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ENG

Lenjoy 1 day comfort

Daily disposable contact lenses with aspheric optics

⚠ Keep this package insert. You may need to read it again.

- It is important that you wear your contact lenses only as prescribed by your eye care professional.
- These lenses are to be worn once during waking hours only and should be discarded at the end of the day.
- Contact lenses are not to be shared.
- Do not use if packaging is not sealed or broken.
- Always wash and dry your hands before handling your lenses.

Lens insertion

- Remove the lens from blister package.
- Place the lens on the tip of your index finger.
- Make sure that the lens is not inside-out. Hold it in front of your eyes and verify the shape (Fig. 1). If the lens is inside-out, gently turn the lens the right way round before wearing it.
- Use the middle finger to pull down the lower eyelid and the index finger of your other hand to pull up your upper eyelid simultaneously.
- Look up at the ceiling and place the lens onto the lower white part of your eye.
- Remove your finger and look down into the contact lens. Blink gently several times to centre the lens.
- Repeat the steps above for your other eye.

Lens removal

- Separate the lids with the middle finger of each hand.
- Look up at the ceiling and with the index finger of the lower hand slide the lens down into the lower white part of your eye.
- Squeeze the lens between your thumb and index finger and remove it carefully.
- The lens should be discarded upon removal.
- Repeat the steps above for your other eye.

How to insert and remove your lenses, see [tutorial on Lenjoy.com](#)

	Supervision Optimax Sdn Bhd Putra Industrial Park Lot 38 Sungai Buloh Bukit Rahman Putra 40160 Malaysia Manufactured with UK Technology		EC Authorized Representative Supermax Healthcare (Europe) Limited 38 Main Street, Swords K67EoAz Co. Dublin, Ireland	Importer Oplea, s.r.o. Videaňská 12 378 33 Nová Bystřice Czech Republic
--	---	--	--	---